Skinial S 400

PIGMENT-ENTFERNUNGS-GERÄT

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1 CONTENTS

Product description / intended use:

This device can be used to remove, and correct unwanted pigmentation caused by permanent make-up, tattooing or microblading. All restrictions and instructions with regard to the treatment explicitly mentioned in the Skinial Training apply. During the treatment, the respective national and local prescribed, hygienic and trade-legal conditions are to be kept. Check with your industry associations and regulators for these conditions are subject to constant change and adjustment.

This device is suitable both for the removal, as well as the attachment of permanent make-up and tattoos. Note the possibly required special requirements and needle qualities.

Technical specifications

Power supply: AC 100-240V / 50-60 Hz

Output: 19V / 2A

Power: 24W

Scope of delivery:

The S 400 product set contains the following components

- 1 x control unit
- 1 x mechanical and magnetic holder for the control unit
- 1 x power supply with EU plug
- 1 x handpiece
- 1 x connection cable
- 10 x Skinial needle 7 RS
- 1 x holder for the handpiece with two perforations for 9 mm ink pots

Optionally, a footswitch is available.

Safety instructions:

- This device should only be used by trained persons who are undergoing a Skinial training in PMU and / or body removal, or who already have a cosmetic or medical education, or have several years of experience in handling pigmentation devices.
- In the case of a resale, this manual should be given to the device.
- This is a professional device, keep it away from children.
- Please use only the original power supply.
- In the event of damage or malfunctions, please send us the device with the proof of purchase to the following address. Do not open the devices yourself and do not carry out any unauthorized repairs, as otherwise the guarantee expires.

Logistic-Mail-Factory GmbH
c / o MIMO GmbH
Affinger Street 7
D-86167 Augsburg

- Improper use, storage or transportation of the controller or handpiece may result in damage. The intended use of the device as specified in this operating manual is the prerequisite for safe use for you and your customers and the basis of the manufacturer's warranty.

2 CONSTRUCTION AND INSTALLATION OF THE CONTROL UNIT

Make sure the adapter, handpiece and footswitch (optional) are properly connected (see assembly instructions below).

Connection of the handpiece

**Connection of the footswitch
(optional)**

**Connection of the
power supply unit**

Assembly of the control unit holder

(1) Remove all parts of the holder kit from the packaging

- (2) Decide whether you want to equip the holder with suction cups (smooth surface) or with the strong magnets (metal surface).
- (3) Mount the three magnets or the suction cups to the bottom of the stand's triangular base plate.
- (4) Secure the controller to the front of the metal frame using the supplied setscrews, which you insert from the rear through the opening in the metal frame and screw into the thread on the back of the controller. Make sure that the control unit always stands on a firm and stable surface.

- (5) Store the remaining parts of the fixture in order to replace the feet in the event of a changed surface.

3 OPERATING INSTRUCTIONS OF THE CONTROL UNIT

- (1) Touch the "POWER" area with your finger for three seconds to turn the controller on or off. (Touch screen)
- (2) Select the present speed by touching the (IOI) button. We recommend "140 / Min. BROW" as standard for PMU and Body treatments. For sensitive skin, the speed can be reduced.
- (3) Touch the "START" area to start the device. By pressing the ">" buttons, you can increase the speed in increments of 10 or reduce it in 10 steps by pressing the "<" button. The speed can also be changed while the device is running.
- (4) Now set the correct needle length on the handset. Turn the thread (see point 4 under instructions / handset) to the desired needle length.
- (5) NPS = approx. Needle speed; Strokes of the needle per second.
- (6) The device can also be used for permanent make-up applications. For Skinial applications, you should only use the BROW and LIP settings. Please make sure to set the correct needle speed.
- (7) A footswitch (optional) can be connected to the "Foot" socket.

In the "Menu", select additional presets, such as

- a. EYE - Eyes (PMU only)
- b. LIP - Lips (Skinial + PMU)
- c. BROW - Eyebrow (Skinial Body & PMU + PMU)
- d. MEDICAL - (medical applications)
- e. MESO - (meso-therapy)

LED display (menu display and speed)

Increase speed

Menu selection


Reduce speed

Start / stop of the handpiece

On / off switch

4 THE HANDPIECE

- (1) Connect the handpiece to the control unit.
- (2) Insert the needle cartridge onto the handpiece. Make sure the cartridge clicks into place.
- (3) By turning the handpiece head, you can adjust the needle length.

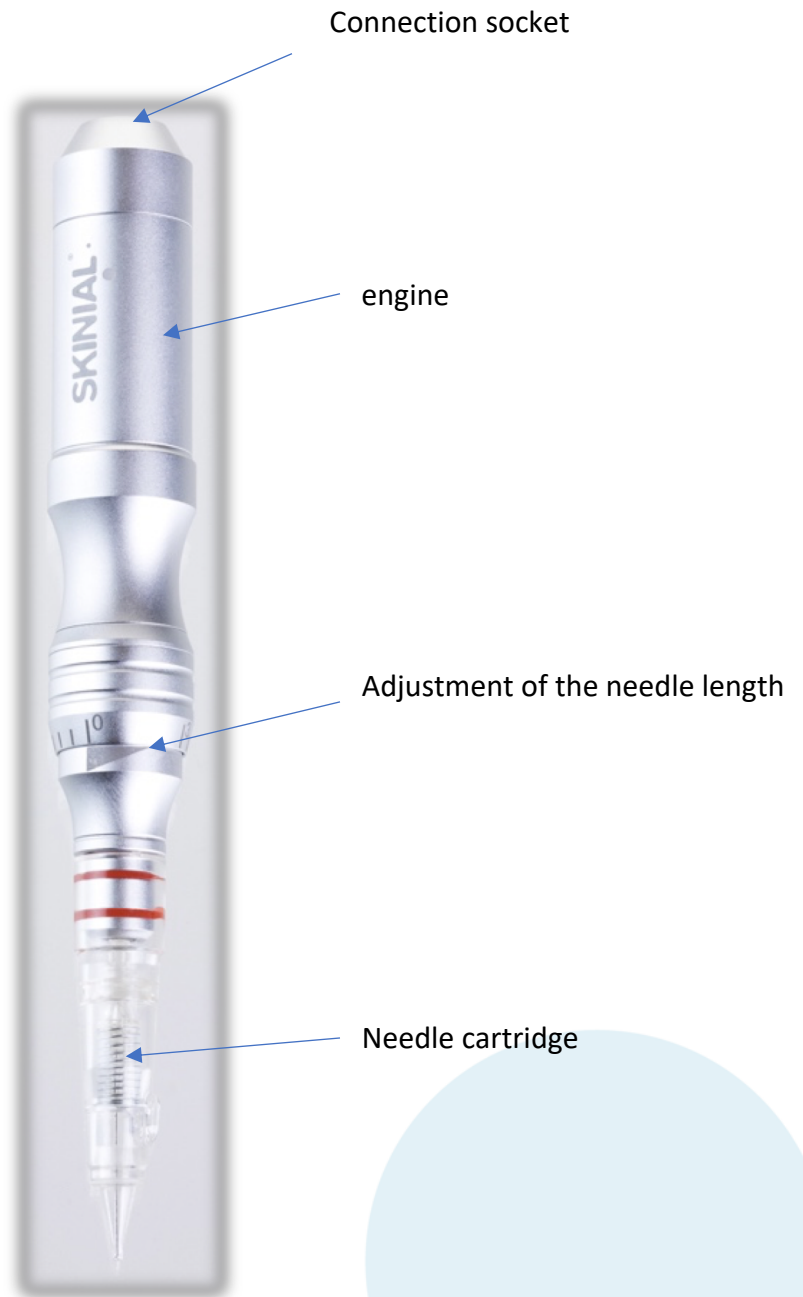Note:

1. The Skinial S-400 handpiece uses a Swiss 12V motor. Under no circumstances should you use a power supply unit with more than 12 V voltage.
2. Use only the original Skinial power supply if possible.
3. To adjust the needle length, start the instrument at a slow speed to see how far the needle protrudes from the cartridge.
4. Turn the front threaded head and correct the needle hub to the desired length against a light background while the device is running. The printed scale is relative, it is not calibrated and therefore cannot be used to set the millimetre.

Adjust needle length by turning

5. Do not over-tighten the needle length adjustment thread as this could damage the handpiece.
6. Regularly check the correct needle length during treatment to detect any accidental changes in good time. Failure to properly seat the cartridge on the handpiece may result in malfunction.
7. Please store and dispose of a used disposable needle in a separate container (hazardous waste) and never use used needles.

5 HANDSET HOLDER

Always place the handset stably and securely in the solid handset holder as shown below (picture). On the support surface and on the bottom are non-slip plastic knobs. Two colour potty wells (9 mm) guarantee a stable filling with the respective Skinial removal liquid.

6 NEEDLE CARTRIDGES

We only use high quality needle cartridges. **ATTENTION: For pigment removal with Skinial only the needle size 7 RS is suitable.** Skinial offers additional cartridge sizes for use with the Permanent Make-Up Hand Tool. Each needle is individually and sterile packed.

Box of 10, single packaging

Precision needle cartridge

7 MISCELLANEOUS

Standards, Laws, Product Labeling, (CE Declaration of Conformity)

This device complies with all necessary legal norms in the EU including the CE Declaration of Conformity at the time of writing. We reserve the right to make changes and technical improvements for the benefit of our customers at any time. If new regulations, laws and regulations lead to a restriction of the operating permit, then the acquirer cannot rely on it and cannot derive any rights from it. However, we will make the necessary changes immediately after knowledge.

Manufacturer and Design:

Skinial AG
Grienbachstrasse 11
CH-6302 Zug, Switzerland
www.skinial.com
info@skinial.com

Country of origin: China

Target group:

This device should only be used by trained persons who have successfully completed a Skinial training in PMU and / or body removal or who already have a cosmetic or medical education or have several years of experience in handling pigmentation devices.

In case of a resale this manual should be given to the device. This device is protected by copyright and may not be copied, opened, modified or altered without the permission of Skinial AG.

Zug, November 2018
Skinial AG